BIOPTRON®

HYPERLIGHT THERAPY SYSTEM By Zepter Group

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This device may become a safety hazard if these instructions/safety precautions are ignored.

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Danger of electric shock! Keep dry! NEVER immerse in water! Do not open!

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Ne bloquez jamais la ventilation.

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Never operate a damaged device (cables, plugs, housing or optical component).

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OPTICAL RADIATION APERTURE • RISK GROUP 2

CAUTION - IR emitted from this device may cause eye irritation. Do not look directly into the light source. If not respected eye damage may occur. For the treatment of the eye area, remove contact lenses and keep your eyes closed.

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For the treatment of newborns use blindfold (PAG-500-BF).

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Repairs and replacements of the halogen bulb should only be conducted by BIOPTRON-authorized service centres! Do not carry out repairs yourself! No modification of this equipment is allowed! Danger if non-genuine substitute parts are used. The expected operational lifetime for B2's device is 5 years and 10 for the floor stand accessory. After that time, a maintenance service should be carried out every two years.

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The power cable is used to disconnect the device from the mains power supply. Do not position the device so that it is difficult to disconnect mains plug.

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The power cable poses a strangulation hazard to children.

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For use in professional healthcare facilities

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In case of unexpected failure or product-related problems while setting up, using, or maintaining the medical device, contact your BIOPTRON Service Center.

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Applications on children and people with cognitive impairment always need to be performed under supervision of a physician.

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BIOPTRON devices can be used on clients with heart implants and pacemakers.

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Medical device is not intended to use in oxygen rich environment and with flammable anaesthetics.

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Contraindications: Before the use of BIOPTRON device consult your physician, especially in the events of: • Photo-allergy • Porphyria (all types) • Actinic reticulosis (photo-allergic contact dermatitis) • Cheilitis exfoliativa actinica (Inflammation of lower lip due to radiation) • Erythema exsudativum multiforme • Hydroa vacciniformis • Xeroderma pigmentosum (extreme sensitivity to UV rays) • Retinal diseases • Taking of photosensitizing pharmaceuticals/herbs.

Potential Contraindications: • Lupus erythematoses • Eye inflammations

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Check with your doctor if BIOPTRON treatment is suitable for you.

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Risk of electric shock. Unplug the device if any liquids or particles enter the device. Contact your service center.

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Unplug the device when cleaning. Protect it from dust. Do not shake it.

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Recyclable packaging. For disposal of the device, follow the local regulations.

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The device must be handled with care as it contains fragile glass elements.

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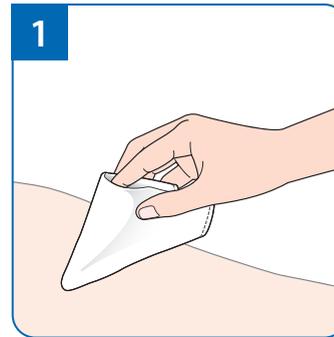
- i** • Do not move during treatment.
- Treat large skin areas one by one.
- Treat areas once or twice a day – see «Medical indications».
- Use preferably in the morning and/or evening, regardless of the indications given.
- Comply with the recommended treatment distance, duration and number of applications per day.
- The device memorizes the last treatment time.
- Never use the device without filter.
- Before and after use, disinfect the ergonomic grip with a soft clean cloth moistened with 70% ethanol.
- The device must be handled with care as it contains fragile glass elements.

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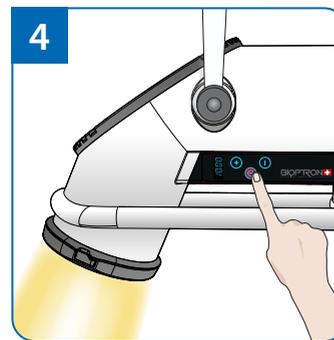
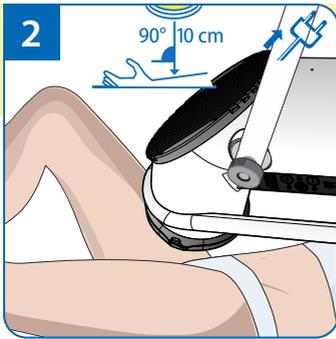
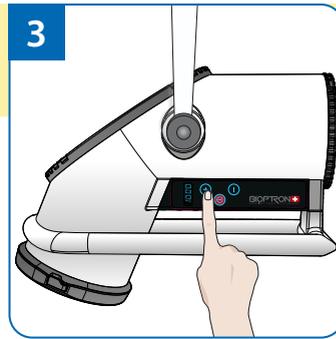
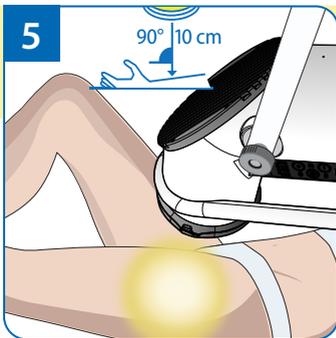
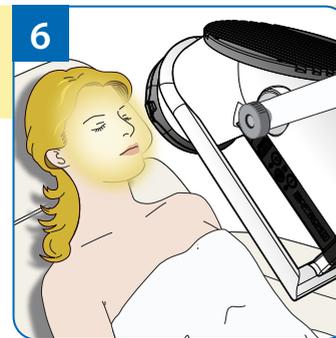
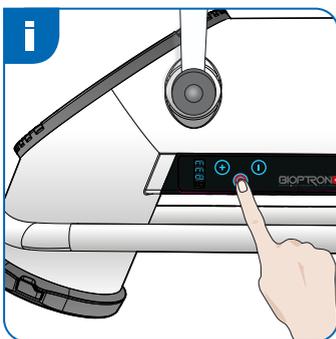
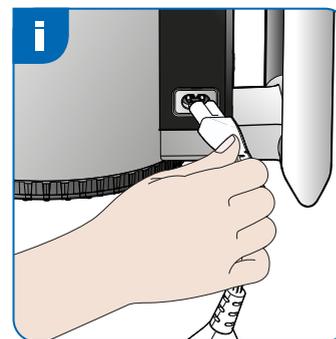
BIOPTRON improves microcirculation • Reinforces the body's defence system • Stimulates regenerative and reparative processes
• Promote wound healing • Relieve pain or decrease its intensity - these basic mechanisms are responsible for the therapeutic healing effects as complementary treatment of the following medical conditions:

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PAIN: recommended treatment time 4 - 10 min, distance 10 cm.

Rheumatology: • Osteoarthritis • Rheumatoid arthritis (chronic) • Arthrosis

Physiotherapy: • Lower back pain • Shoulder and neck pain • Carpal tunnel syndrome • Scar tissue • Musculoskeletal injuries

Sports medicine – soft tissue injuries of muscles, tendons and ligaments: • Muscle spasm • Sprains • Strains • Contusions • Tendonitis • Desmorrhexis and myorrhexis (ligament and muscle tears) • Epicondylitis (tennis elbow)

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WOUNDS: recommended treatment time 2 - 8 min, distance 10 cm.

• Wounds due to traumas • Burns • Grafts • Wounds after surgery
• Venous leg ulcers (stasis ulcers) • Decubitus (pressure) ulcers

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DERMATOLOGICAL DISORDERS AND SKIN PROBLEMS:

recommended treatment time 4 - 10 min, distance 10 cm.

- Atopic dermatitis • Psoriasis • Herpes simplex and zoster • Surface bacterial infections • Acne • Acne rosacea

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IN PEDIATRICS: APPLICATIONS ON CHILDREN UNDER 6 YEARS OLD MUST BE PERFORMED UNDER

SUPERVISION OF A PHYSICIAN. Recommended treatment time 4 - 10 min, distance 10 cm.

- Pediatric dermal affections • Endogenous eczema • Upper respiratory tract infections • Allergic respiratory diseases • Pediatric musculoskeletal and neurological disorders and deficits

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IN NEWBORNS: APPLICATIONS MUST BE PERFORMED UNDER SUPERVISION OF A PHYSICIAN:

Do not apply in eye area and use blindfold. Recommended treatment time 4 - 10 min, distance 10 cm.

- Dermal affections as phlebitis, decubitus, intertrigo.

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- Typical winter depression symptoms such as chronic fatigue syndrome, lowered motivation, hypersomnia, asthenic conditions, break down.

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B2 Floor stand
B2 Bodenstativ
Pied de sol B2
Piedistallo B2

Put the B2 floor stand on a firm and flat surface. Hold device tight while adjusting. Use only original BIOPTRON stands. The manufacturer refuses any liability if other stands are used!

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 **RISK GROUP 2: EN 62471: 2008**
CAUTION Do not stare at the light source
CAUTION IR emitted from this device may cause eye irritation. 
 Specific Power Density: 3.8W Degree of Polarisation: >95% 590 – 1550 nm

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 **2019-08** **CE 0197** **IP20**     

The recommended minimum distance of the device B2 to portable and mobile RF communication equipments depends on their transmission power and operation frequency. The distance can be taken from the below table.

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For transmitting powers which are not shown here, the distance can be calculated with following formula:

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